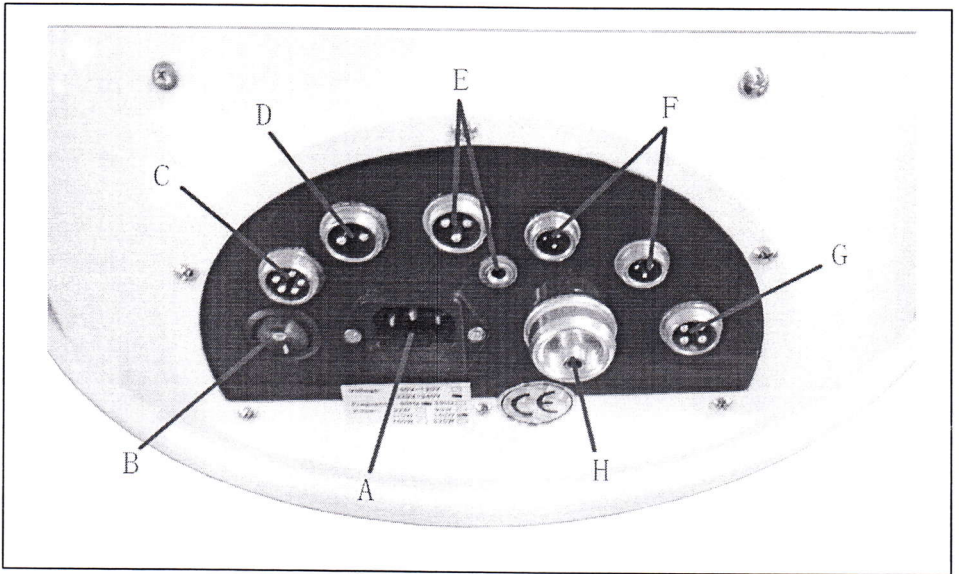


NV-Q607 7 Functional Diamond Dermabrasion

7 Functions

- Diamond Dermabrasion
- Ultrasonic
- Photon
- Magic Glove
- Cold & Hot Treatment
- Skin Scrubber
- BIO

Connections



A— Power Socket

B—Power Switch

C—Photon Therapy Socket

D—BIO Face Lift and Magic Glove

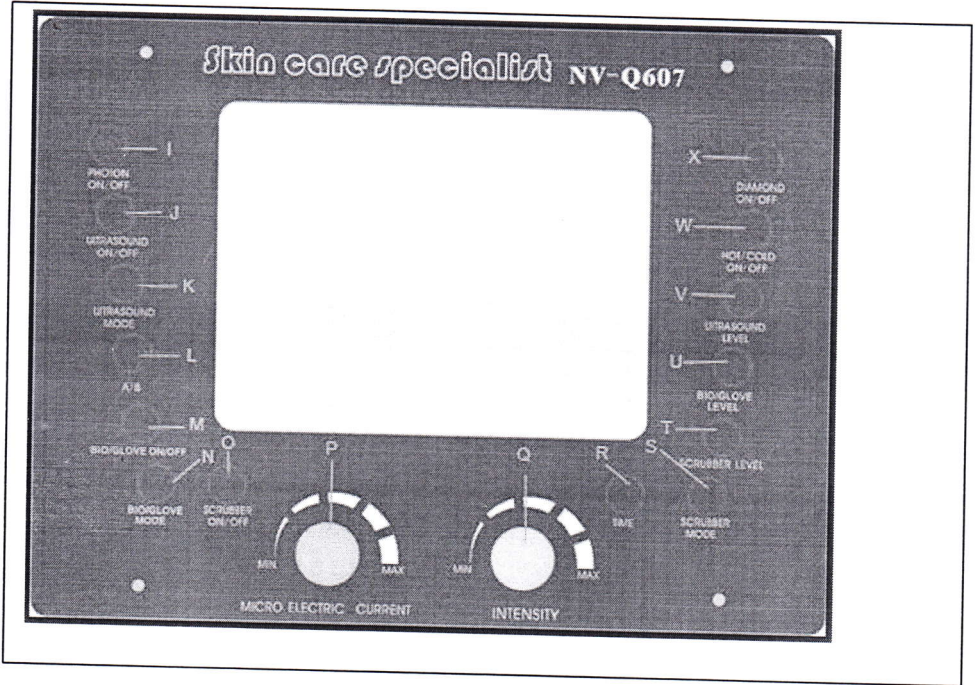
E—Skin Scrubber , the Neagtive Stick Socket is not available now

F—Ultrasound Socket

G—Cold&Hot Hammer

H—Diamond Vacuum Output

Face Panel Instruction



- I-- Photon Therapy Power Switch
- K— Ultrasound mode
- M— BIO &Glove Power Switch
- O—Skin Scrubber Power Switch
- Q—Diamond Vacuum Intensity
- S—Skin Scrubber mode
- U—BIO & Glove Intensity
- W—Hot&Cold hammer Power Switch

- J—Ultrasound Power Switch
- L—Ultrasound Probe A/B
- N—BIO & Glove Mode
- P—Micro Current Intensity
- R—Time
- T—Skin Scrubber Intensity
- V—Ultrasound Intensity
- X—Diamond Power Switch

For Function & Operation Details please refer to:

Contents/ Function & Treatment/ ①②③④⑤⑥⑦

1- Finish all accessories connection before plug the power cord.

Don't plug the accessory power wires to the sockets when the machine powers are on.

2- Be sure the Main Power Button on the machine back side is on→ Set the "TIME"

● **Diamond:** Press DIAMOND ON/OFF to start→ Adjust the Intensity Knob for the suction you want→ Press DIAMOND ON/OFF to stop after use.

● **Cold & Hot:** Turn on the panel power "HOT & COLD"→ Press the power button on the hot & cold hammer→ Turn off both power buttons after use.

● **Ultrasonic:** Set TIME and adjust the INTENSITY→ Press Ultrasound ON/OFF to start, choose "Pulse/Continuous Wave (Ultrasound Mode)"→ Press ON/OFF to stop after use.

- OUTPUTS "BA": To control the 2 probes of Ultrasonic

The ultrasound flat probe & pointed probe cannot work at the same time. (Choose B or A for exchange)

● **Skin Scrubber:** Set TIME and adjust the INTENSITY→ Press scrubber ON/OFF to start, choose "Pulse/Continuous Wave(Scrubber Mode)"→ Press ON/OFF to stop after use.

- When operating Ultrasound or Skin Scrubber, let the client hold the **Negative Current Handle** and then operator hold the Ultrasound/ Scrubber and working on the face.

Caution: Please don't turn on usage with long time, better use **Scrubber** within 15~30 minutes per time. Then stop the scrubber for 20 minutes before the next scrubber operation. Long time operation will burn the scrubber grey box and destroy the control board of the whole machine.







- **Photon:** Be sure the Main Power Button on the machine back side is on
→ Contents/ Function & Treatment/⑤
 - **BIO&Magic Glove:** Be sure the Main Power Button on the machine back side is on→ Contents/ Function & Treatment/⑥
- 3- After Operating, turn off the power buttons and disconnect the power cord, then disconnect and clean accessories.

① Diamond Microdermabrasion Function

※ Function

The Diamond Microdermabrasion provided a non-surgical skin refinish procedure, by using sterile diamond heads to abrade or rub off the top skin layer, then Vacuuming out the particles along with any dirt and dead skin back up. This procedure removes skin debris, imperfections, blemishes, wrinkles and unwanted pigmentation on the skin. Moderate usage of Diamond Microdermabrasion, accompany with skin products that penetrate through top skin layer that reach the dermis level, helps to replenish natural nutrients, restore cell activity, and promote skin health. The results after Diamond Microdermabrasion treatment is usually a healthy, glowing, beautiful skin.

※ Accessories


| | | | |
|---|---|---|---|
|  |  |  |  |
| PVC hose × 1 | Rubber rings & Fuses | Big cotton filter × 1 pack | Small cotton filter × 1 pack |
|  |  | Stainless steel wands × 2 Diamond tips × 9 | |
| Color Box → open | | | |

※ Preparation before treatment

1. A consultation included the examination of the skin, a plan for treatment.
2. Operator should wear vinyl gloves and a facemask.
3. On oily or acne skin, a steamer may be used prior to the treatment, or a degreasing lotion may be applied prior to the cleanser.
4. Cleansing the skin with Cleansing gel or water, then wipe off with cotton pad. Allow the skin to dry.

※ Operating Instruction

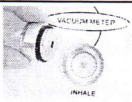1- Please first connect all accessories to the machine.



1) Unscrew the vacuum connector, make sure a big cotton filter is in the center & a large black ring is around connector head.



2) Screw down the connector after check. Then attach the PVC hose to the connector.



3) Place the other end of PVC hose in the end of wand.



4) Place a small cotton filter on the head of wand. (see, a small black ring is around the head)



5) Screw the diamond tip into the wand (choose proper size tip for customer, see behind **diagram of tips set**)

2- Put the plug into the power socket and turn on the power button. Turn the intensity knob to left side (minimum).

3- Adjust the vacuum intensity slowly from minimum to maximum, the vacuum meter will show the data of pressure. For commonly use, do not set the intensity to maximum unless necessary.

Recommended Vacuum intensity on treatment area

| Treatment Area | Normal Skin (inHg) | Sensitive Skin (inHg) |
|----------------|--------------------|-----------------------|
| Face | 12-22 | 8-12 |
| Forehead | 12-22 | 10-12 |
| Chest | 15-20 | 15-18 |
| Nose | 10-15 | 10-14 |
| Neck | 18-18 | 10-15 |
| Near Eye Area | 8-12 | 8-10 |

4- Treatment Schedule Table

| Treatment | Frequency (days) | Total Visits |
|-------------------|------------------|--------------|
| Face Peeling | 6-10 | 3-5 |
| Fine Lines | 7-13 | 6-8 |
| Neck | 12-14 | 3-5 |
| Acne Scars | 7-12 | 8-10 |
| Hyperpigmentation | 7-12 | 8-10 |

5- Place the diamond head on skin surface (start from forehead), proceed with a slow sliding motion. Do not leave the diamond tip at one spot for too long, doing so will cause harm to the skin.

6- Gently stretch the skin with thumb and finger and glide the Hand-piece 2 to 3 times over the area being treated. Then cross again in different directions to prevent streaking lines as in the reference picture. Brush strokes or polishing motions may be used in different sites. (See behind **diagram of suggested movements and areas**)

7- After treatment, brush off all remaining dead skin with a soft brush and rinse with damp sponges or damp cool towel.

8- Allow skin to dry then apply desired skin product.

9- To clean the Diamond Head, use toothbrush or cotton swab damped with a little alcohol to wipe off the dirty impurity, and then use UV light or high temperature for sterilization.

DIAGRAM OF SUGGESTED MOVEMENTS
AND AREAS FOR DERMABRASION

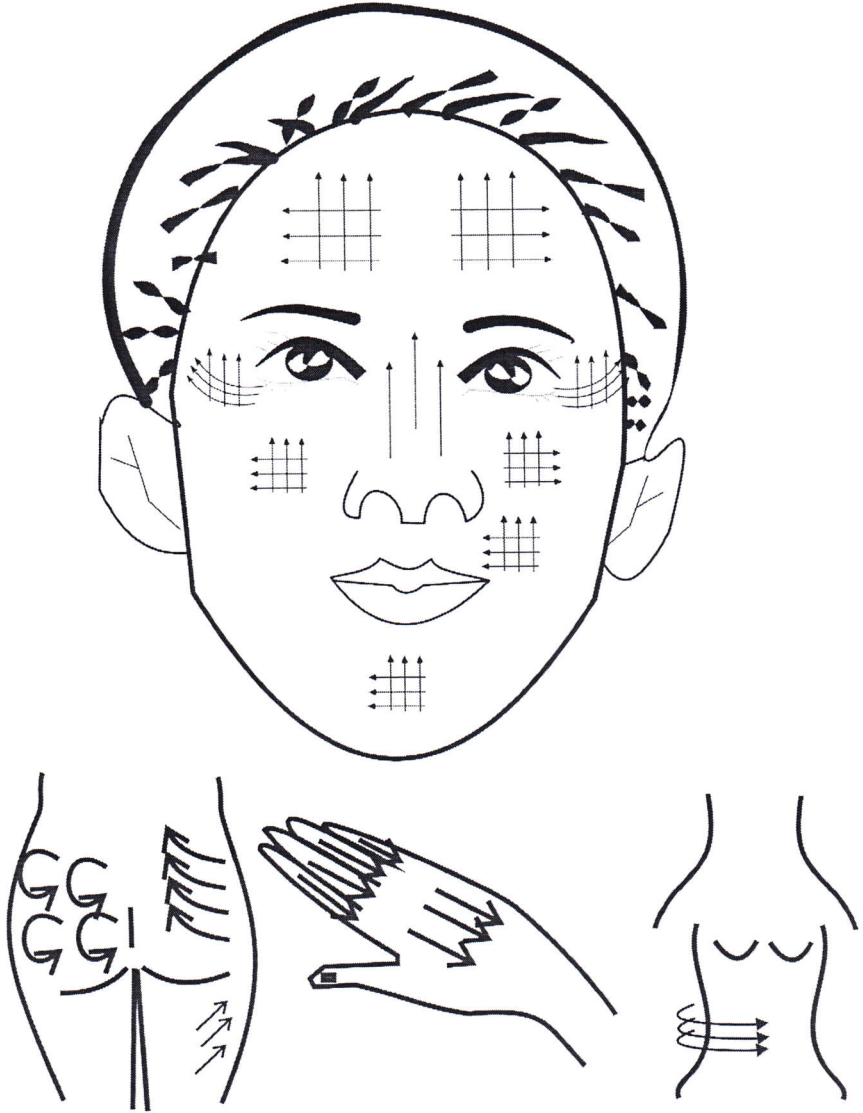




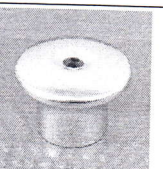


Diagram of Diamond Dermabrasion Tips & Wands Set

2 stainless steel wands which are used with 9 interchangeable diamond tips.

| | | | |
|---|--|------------------|---|
| 1 |  | D80 | Body. Especially for acne, knee, elbow areas and striae of pregnancy. |
| 2 |  | D100, D120 | Face. For the larger blemishes. |
| 3 |  | D150, D180, D220 | For Scars, fine lines and imperfections. |
| 4 |  | D160, D180 | Used on the eyes and both sides of nose. For wrinkles, oil tablets, blackhead and acne. |
| 5 |  | Smooth Tip | After the D tips peeling procedure, use this smooth tip to vacuum/clear away the dead abraded skin. |

- 1- The higher the D number, the finer the grit.
- 2- Clean diamond tips with Alcohol, then sterilize them with UV light or high temperature.
- 3- Replace the big cotton filter inside the vacuum connector after every five to six treatment.
- 4- Replace the small cotton filter on the wand head after every treatment.

※ Safety Warning

Never operate this equipment on a person that suffers form the following:

- a. Heart Disease
- b. Sensitive skin
- c. Damaged skin
- d. Inflamed skin
- e. Infected skin

Do not use on the following areas:

- 1. Sensitive skin
- 2. Skin rash
- 3. Damaged, inflamed, or infected skin
- 4. Surgical scar that is less than 12 months old
- 5. Swollen skin
- 6. The probes should not point directly towards the eyeball
- 7. Do not use on the genitals

② Cold & Hot Function

※ Function

The cold treatment is specially designed to calm down the skin after diamond microdermabrasion or chemical peel. The skin is sensitive after peeling treatment, so the cold treatment procures is necessary. The hot treatment is designed for the acceleration of product absorption.

※ Accessories



Cold/Hot Hammer × 1

※ Operating Instruction

- 1- Connect the Cold and Hot hammer to the “Cold & Hot” socket.
- 2- Set microdermabrasion/ultrasound timer then press the “Hot/Cold” hammer button (according to panel). You will hear the fan turn on.
- 3- To start the cold function press the button on the “Hot/Cold” hand piece down (-). The Metal on the cold hammer side will get very cold.
- 4- To start the hot function press the button on the Hot/Cold” hand piece up (o). You will only hear the fan speed up. The other side of the hammer will get hot, but not extremely hot.
- 5- Wait 2-3 minutes for the hammer to get cold or hot before starting treatment.
- 6- The treatment should be approximately 5-10 minutes. Do not stay in one area of the skin too long this will avoid any damage to skin from the hot/cold head.
- 7- Press the Cold/Hot button on panel again to turn off.
- 8- Clean head before and after every treatment, wiping with alcohol or a disinfectant wipe.

③ Ultrasonic Function

※ Function

Ultrasonic utilizes sonic vibrations; it works one million times per second (1 MHz). The ultrasonic wave could stimulate skin tissue and improve permeability of the membrane to promoted collagen tissue, reduced wrinkle and invisible appeared fine line, also lift the skin result for younger and healthy look. Ultrasonic generate frictional heat which

accelerate blood circulation, muscle relaxation, and decompose the cellulite to break down exceed fat.

※ Accessories



Face Electrode (flat head) × 1

Eye Electrode (pointed head) × 1

(Remove the light green rubber covers before use)

※ Preparation before treatment

- 1- Before beginning Ultrasound make sure the skin is cleansed, and then apply an appropriate gel, serum, or cream for the appropriate skin type or condition. Make sure the product has some slip to it.
- 2- Choose the pointed or flat head. Use pointed head for around the eyes, and the flat head for the face and neck.

※ Operating Instruction

- 1- Connect the face/eye electrode into the ultrasonic electrode outlet. Make sure the machine main power button and panel ultrasound power button are on.
- 2- Set the ultrasound timer (and press “Wrinkle” button if has it)
- 3- Press the intensity level button “up” or “down” for intensity desired. Intensity levels are 0-9, from minimum 0 to maximum 1.8 Watts/cm. Each level is 0.2 Watts/cm. For Facial work, low to mid intensity is recommended (0-4). For Body work, mid to maximum intensity is recommended (5-9).
- 4- Press pulse or continuous wave indicator light.
- 5- Press the button Start or >/■. (according to face panel)

For some exceptional machines (such as NV-B604, B605 & 666), let the client hold the Negative Current Handle, the operator hold face/eye Electrode and working on the face.

※ **Safety Warning**

- 1- Do not treat eyelid directly. Treat just below the eyebrow and brow bone.
- 2- Never stop the electrode on the skin in one spot.
- 3- Always keeps the electrode moving across the skin.
- 4- Never point the electrode towards chest area.
- 5- Clean lightly the popping head with cotton. Do not rinse it with water. Be careful not to drop or crash.
- 6- Pregnant, patients with disease of blood vessel, heart, skin, bleeding & trauma are prohibited to use ultrasonic.
People with abnormal kidney, liver disease are not suitable for this function.

④ **Skin Scrubber Function**

※ **Function**

Scrubber function removes dead skin cells, smooth wrinkles, treats pimples and clears the complexion. It uses ultrasonic waves to shake loose dead skin cells, it also create heat and vibration to give a pleasant massage which will drive creams and ampoules into the skin.

※ **Accessories**



Scrubber Handle × 1

Negative Current Handle × 1

※ **Operating Instruction**

Caution:

- a) Please don't turn on usage with long time, better use scrubber within 15~30 minutes per time. Then stop the scrubber for 20

minutes before the next scrubber operation. Long time operation will burn the scrubber grey box and destroy the control board of the whole machine.

2- If saloon treatment schedule is full everyday, please prepare two or more machines for alternate use. Don't use one machine for several customers and never stop, this will reduce the service life of machine.

- 1- Attach the Skin Scrubber & Negative Current Handle to the outlets, (make sure to remove cover before use).
- 2- Make sure the main power buttons are on.
- 3- Press the skin scrubber timer button, then set the skin scrubber intensity using the "up" or "down" buttons. Always consider the clients comfort level. You can increase the intensity when needed.
- 4- Press the skin scrubber "on/off" button. You will hear the scrubber start to "tick"
- 5- Choose pulse or continuous current according to your needs. You will hear the difference between pulse and continuous current.
- 6- Let the client hold the Negative Current Handle and then operator

hold the Scrubber.

Cleaning Skin

- 1- Apply cleanser to skin. Massage well and remove using the skin scrubber with the curved side pointing up, at a 45 degree angle (removing product, with a slight scraping motion). Wipe any excess product on a tissue or a cotton 4x4 pad. Skin Scrubber should be on continuous current for this process.
- 2- Finish removing cleanser with sponges, cotton rounds, or hot/warm towel.

Skin Care

- 1- After the skin is properly cleansed, apply a serum, cream, gel or ampoule appropriate for the skin type and skin condition of your client.
- 2- Using the small flat side of the skin scrubber, move around the skin to penetrate product. The skin scrubber should be set to the pulsating current.

※ Safety Warning

- 1- Never operate this equipment on a person that suffers from the following:
 - a. Heart Disease
 - b. Sensitive skin
 - c. Damaged skin
 - d. Inflamed skin
 - e. Infected skin
- 2- Use proper product for the result you are trying to achieve. Do not use on skin without product.
- 3- Never allow the blade of the Skin Scrubber to sit on one area of the skin for a long period of time, keep it moving over the skin.
- 4- Do not use on eye area.
- 5- Do not immerse the handle or machine in water
- 6- After each use, clean the scrubber blade with alcohol or disinfectant and store in a cool, dry place.

⑤ Photon Function

※ Function

Senility-resisting, anti-allergic introducing nutrition, loosing skin, degreasing, improving skin, activating cell, reducing inflammation, accelerating cell reproduction and cell membrane reproduction, written the skin, reducing spot and melanin.

※ Accessories



Photon Probe × 1

※ Operating Instruction

1-It is not recommended to put the probes on a same position for longtime. Probes may move on the skin in regular line or circle.

| | |
|--|---|
| | <ul style="list-style-type: none"> • WAVE <p>There are 2 kinds of waves to choose, Continuous Wave & Intermittent Wave</p> |
| | <ul style="list-style-type: none"> • MODE/ON <p>MODE: Choose different color (among 7 colors) ON: Start the photon function</p> |
| | <ul style="list-style-type: none"> • AUTO <p>Those 7 colors will operated automatically .</p> |
| | <ul style="list-style-type: none"> • OFF <p>Stop the photon function</p> |

2-Choose the proper spectrum reaching different depth on skin.

| Orders of color by pressing MODE/ON button | Depth In Skin (Max 6, Min 1) | Suitable Treatments For |
|--|------------------------------|---|
| 1 Red | 6 | Aging, Loose, Lead In Nurture, Burn Fat |
| 2 Green | 4 | Capillary vessel dilation, Splash, Detumescence, Diminish inflammation, Eye lifting, Dark eye rim, Pouch, Face, Light speckle, Breast lifting |

| | | |
|----------|-----------------|--|
| 3 Blue | 2 | Acne, Susceptivity skin, Diminish inflammation and pox |
| 4 Yellow | 5 | Dim skin, Lark skin, Improve lymph, Activate cells |
| 5 Purple | 1 | Accelerate cell regeneration |
| 6 Cyan | 3 | Anti hypersensitivity, Reducing appearance of capillary vessel |
| 7 White | 1~6 Combination | Whitening, Light speckle, Reducing melanin |

Note:

- Before use, choose color according to the skin status.
- The unsuitable setting may cause rubefaction on sensitive skin. When this occur, can use the cold side of Cold & Hot Hammer to cool down the area. The rubefaction will disappear in 24 hours.

Photon Function can be combined with "**MICRO ELECTRIC CURRENT**", which will make the photon function more effective

- i. The Micro Current should be **MIN** at the beginning, then adjust it stronger according to customer's needs.

⑥ & ⑦ BIO Function

※ Function

Skin tightening after cellulite and fat breaking down;
Lifting up, for example, faces, breasts, hips and etc.

※ Accessories



Spot handle × 1 pair
Mushroom handle × 1 pair
Handles without Cap × 1 pair
Magic Glove × 1 pair



※ Operating Instruction

1. Connect one pair handles to BIO outlet.

There are 3 pairs different handles available:

Mushroom handles: For large area such as the cheeks, forehead, neck, etc.

Spot handles: For small area, such as nasal fossa, and so on.

Handles without Cap: Please insert the Cotton stick (which you can find them in the manual book plastic bag), then dampen the cotton with anti-wrinkle essence or other kind of essences.

Note: Handles with damp cotton stick conducts electricity stronger than the other 2 kinds of handles.

2. Select the time as you like about 15 minutes or 30 minutes.
3. Select the strong or weak intensity. Adjust the BIO intensity slowly from minimum to maximum, for commonly use, do not set the intensity to maximum unless necessary.
4. Select the mode just you like such as hip, bust, neck or face

Can be used on:

- Micro wave Anti-wrinkle massage suitable for sagging, ageing, excrescence skin and eye bag muscle.
- Skin Maintain the pores and vessels. Clean up pores and vessels. Close up pores tighten muscles. Improve nutrition absorption and permeability. At the same time, prevent aging and help face reforming.
 - Body Purify our body and blood. Smash fat cells and burn the fat have a same effect of arranging then makes our body look more airy-fairy.

Package Included

- Diamond Dermabrasion Machine X 1
- PVC vacuum hose 6*4 X 1
- Stainless steel wands X 2
- Diamond tips X 9
- Cotton filter X 2 packages (1big, 1 small)
- Ultrasonic probe X 2 (flat and pointed)
- Cold/Hot Hammer X 1
- Skin Scrubber Handle X 1
- Negative Current Handle X 1
- Photon Probe X 1
- BIO Spot handle X 1 pair
- BIO Mushroom handle X 1 pair
- BIO Handles without Cap X 1 pair
- Glove X 1 pair
- One-time insulating gloves X 3pair
- Rubber rings and fuses
- Full detailed English manual/ instructions

Specifications

Voltage: 220V - 240V/50Hz □
100V - 120V/60Hz □

Power: 140W **Fuse:** 3A

Packing Size: 49.5* 38.5* 29cm **G.W:** 11kg

NEWFACE PABX
KingsonCALLERIDCOM-86

Warranty Card (No.):

Product's Name

Date of Purchasing

Customer's Name

Customer's Address

Customer's Telephone

Post Code

E-mail

Warrant explains

1、 We warrant the product to be free for one year from the date of purchasing. If your system has some failure when you in normal using, you can contact the agent who sell you the product. Please show this card when you ask the service, our agent has the right not to supply the service if you don't have this card.

*2、 The following situations cannot be free maintenance.
There is no effective bill*

Because of wear and tear normally caused by the periodicity check, safeguard, maintenance, or spare parts changing.

Because of fall, push, soak water, moist and other damages caused artificially.

The damage of natural calamity: floods, fire, lighting stroke etc.

The damage caused by not using according to operation introduction.

Dismantle and install the manufacture No. of the product had been changed, cancel, move or slur.

The above arrange if had changed, is subject to new regulation of our company.